

HOW TO CLEAN AND DISINFECT SURFACES USING VIRACLEAN®



Viraclean® benefits:

- ✓ Preferred hospital grade disinfectant in Australia and New Zealand
- ✓ is non flammable
- ✓ is suitable for use on healthcare surfaces
- ✓ does not contain chlorine or oxidising biocides
- ✓ will not cause staining on fabrics or carpets
- ✓ has a neutral pH
- ✓ is friendly to metals (non corrosive)
- ✓ has a mild lemon fragrance
- ✓ suitable for use in both dirty and clean conditions

APPLICATION:

USE UNDILUTED FOR DISINFECTION

The use of gloves is recommended whenever using disinfectants.

To remove organic soiling (blood, sweat, sputum etc) from a surface, it is recommended to apply Viraclean® liberally around the soiled area and work towards the centre. Soak up using fresh, disposable cloth.

Once clean, to disinfect the surface, apply undiluted and leave to dry for 10 minutes at room temperature. If required, soak up using fresh, disposable towelling. Used towelling should then be disposed to a suitable receptacle. Not intended for use on therapeutic devices.

STORAGE: Store below 30°C out of direct sunlight.

SPILLS: Absorb with inert material and dispose of into a suitable waste receptacle. Wash any remnants to sewer.

FIRST AID: Contact a Poisons Information Centre (Phone: Australia 131 126, New Zealand 0800 764 766).



HOW TO CLEAN AND DISINFECT SURFACES USING VIRACLEAN®



Step 1: Personal Protective Equipment.

Ensure appropriate PPE is worn prior to commencing.



Step 2: Clean and Remove Soiling.

To remove organic soiling from a surface, apply Viraclean® liberally around the soiled area and work towards the centre.



Step 3: Disinfecting.

To disinfect the surface, apply undiluted.



Step 4: Allow surface to dry.

Leave to dry for 10 minutes at room temperature.

Viraclean® Uses:

Nursery/Childcare:	Cleaning cribs, cots, change tables.
CSSD:	Bench tops, stainless steel surfaces, sinks, trolleys, tap fittings.
Bathroom (toilet/shower):	Sinks, tile surfaces, and all hard surfaces.
Intensive Care Units:	Cleaning and disinfection of all hard surfaces.
Manual Cleaning of Theatre Equipment: (excludes medical devices)	VIRACLEAN® can be used to clean and disinfect equipment, cables and will not adversely affect surfaces.
Operating Theatres:	Cleaning and disinfecting theatre surfaces.
Blood Spills: (excludes medical devices)	VIRACLEAN® is ideal for cleaning and disinfecting blood spills.
Telephones:	VIRACLEAN® is ideal for cleaning and disinfecting phones where large numbers of bacteria are found.
Dental & GP Surgeries: (excludes medical devices)	VIRACLEAN® can be used in Dental & GP Surgeries for cleaning and disinfecting surfaces including chairs, stools, tables, benches and light fittings.
Laboratories & Pathology Clinics:	VIRACLEAN® is excellent for wiping down benches, disinfection of white rooms, cleaning and disinfection of fume cupboards, etc.

The Viraclean® label lists that it is proven to kill:

- ✓ Acinetobacter
- ✓ Candida albicans
- ✓ Coronaviruses including SARS-CoV-2 (COVID-19)
- ✓ Enterococcus faecalis (VRE)
- ✓ Escherichia coli (E coli)
- ✓ Hepatitis B Group virus
- ✓ Herpes Simplex virus
- ✓ Influenza virus
- ✓ Klebsiella pneumoniae (CPE/CRE)
- ✓ Proteus vulgaris
- ✓ Pseudomonas aeruginosa
- ✓ Salmonella choleraesuis
- ✓ Staphylococcus aureus (MRSA or Golden Staph)

