

# VIRACLEAN

## HOSPITAL GRADE DISINFECTANT

AUST R 69000  
(Patented)

### ABOUT VIRACLEAN:

**VIRACLEAN** passes TGA Option B and kills a broad range of other bacteria including Enterococcus Faecalis (VRE), Acetobacter and Acinetobacter and is also proven effective against Hepatitis B Virus, Herpes Simplex Virus and the Influenza Virus. **VIRACLEAN** is a major development in advanced cleaning and disinfecting technology from Whiteley Medical. It is the result of years of intensive research and development. A considerable investment in both time and money has been made in the interests of improving the safety and efficacy of hospital grade disinfectants.

The revolutionary hepadnacial test method was developed at the University of Sydney sponsored with funding from Whiteley Medical and represents another world breakthrough in Australia. This glutaraldehyde-free product, **VIRACLEAN**, which was developed during this programme, has been patented.

### MICROBIOLOGICALLY TESTED:

**VIRACLEAN** has been *proven* to kill:

- ✓ Staphylococcus aureus (MRSA or Golden Staph)
- ✓ Pseudomonas aeruginosa
- ✓ E coli
- ✓ Enterococcus Faecalis (VRE)
- ✓ Acinetobacter
- ✓ Acetobacter
- ✓ Salmonella choleraesuis
- ✓ Proteus vulgaris
- ✓ Influenza virus
- ✓ Herpes Simplex virus
- ✓ Hepatitis B Group virus

### PRODUCT DESCRIPTION:

**VIRACLEAN** is a pink liquid with a lemon fragrance. It is easy to use and is conveniently packaged in squeeze or spray bottles, 5 litre containers, and 15 litre containers. This product is not for use on registered medical devices. Instrumax Pink is available as an instrument disinfectant low level for use in disinfecting non critical medical devices.

Viraclean - Page 1 of 3  
0508



**CHEMICAL SPECIFICATIONS:**

Surfactant Blend.....	Blended
Colour.....	Light Pink
Colour of Dilute Solution.....	Colourless
pH.....	Neutral (7)
Solubility.....	100% in all water
Environmental.....	Completely biodegradable
Specific Gravity.....	0.99
Corrosion Inhibitor.....	Not necessary
Foaming Properties.....	Slight
Abrasives.....	Nil
Free Caustic.....	Nil
Phosphates.....	Nil
Flash Point.....	Nil
Occupational Safety.....	Use as spray or liquid
Dangerous Goods Rating.....	Not applicable
Soap.....	Nil
Shelf Life.....	18 months
Safety Equipment.....	Gloves recommended

**PRODUCT FEATURES:**

**VIRACLEAN:**

- ✓ is non flammable.
- ✓ is suitable for use on healthcare surfaces.
- ✓ does not contain phenolics, chlorine or glutaraldehyde.
- ✓ will not cause staining on fabrics or carpets.
- ✓ has a neutral pH.
- ✓ is friendly to metals (non corrosive).
- ✓ is fully biodegradable.
- ✓ can be used neat for use as a hospital grade disinfectant.

Some of this data has been internationally published.<sup>1</sup> **VIRACLEAN** passes the TGA option B (dirty conditions). This independent data is available on request.<sup>2</sup>

**VERSATILE:**

**VIRACLEAN** is formulated to replace most cleaning products and hospital grade disinfectants. **VIRACLEAN** can be used for cleaning and disinfecting.

**HELPS CONTROL INFECTION:**

**VIRACLEAN** is the only solution that can be used for routine cleaning and can also be used for disinfecting where necessary. This makes for simplification of cleaning and disinfecting procedures.

**PRODUCT USE:**

<b>Nursery:</b>	<i>Cleaning cribs, cots, change tables.</i>
<b>CSSD:</b>	<i>Bench tops, stainless steel surfaces, sinks, trolleys, tap fittings.</i>
<b>Toilet Shower:</b>	<i>Sinks, tile surfaces, and all hard surfaces.</i>
<b>Intensive Care Units:</b>	<i>Cleaning and disinfection of all hard surfaces.</i>
<b>Manual Cleaning of Theatre Equipment:</b>	<b>VIRACLEAN</b> can be used to clean and disinfect equipment and cables and will not adversely affect surfaces.
<b>Operating Theatres:</b>	<i>Cleaning and disinfecting of tables in theatre.</i>
<b>Blood Spills:</b>	<b>VIRACLEAN</b> is ideal for cleaning and disinfecting blood spills on all hard surfaces.
<b>Telephones:</b>	<b>VIRACLEAN</b> is ideal for cleaning and disinfecting phones where large numbers of bacteria are found.
<b>Dental &amp; GP Surgeries:</b>	<b>VIRACLEAN</b> can be used in Dental & GP Surgeries for cleaning and disinfecting surfaces including chairs, stools, tables, benches and light fittings.
<b>Laboratories &amp; Pathology Clinics:</b>	<b>VIRACLEAN</b> is excellent for wiping down benches, disinfection of white rooms, cleaning and disinfection of fume cupboards, etc.

**ENVIRONMENTAL STATEMENT:**

Whiteley Medical manufactures products which comply with Australian and International environmental policy. Whiteley Medical aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley Medical products are supplied in 100% recyclable containers.

Whiteley Medical and its' staff are fully committed to ensuring that our Australian environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

**REFERENCES**

I. International Journal of Epidemiology and Infection Control (1991) Vol. 106, pp 435-443.

**PACKS:** 12x500 mL Squeeze Pack, 12x500 mL Spray Pack, 2x5 litre carton, 15L.

© This document remains property of Whiteley Corporation Pty Ltd.  
Alterations are not permitted without prior written authorisation from  
Whiteley Corporation Pty Ltd.